



Packing Instructions

For Continuous Passive Motion Machine

Manufactured by:
Ortho Rehab
Otto Block Health Care
Ontario
Canada

Distributed by:
CPM Knee Hire
Stockport
England

PACKING

It is imperative these instructions are followed exactly otherwise the machine will not fit into the box.

- Release the yellow knobs and slide the foot plate out then re-tighten.
- Wrap the foot plate with provided bubble wrap
- Drive the machine to **0 degrees**.
- Release the red knobs at the rear of the machine **set to 30** and re-tighten.
- Switch off.
- Remove the mains cable from the power supply.
- Remove the keypad and put into small inner box (wrap in provided bubble wrap).
- Remove the power supply and put into small inner box (wrap in provided bubble wrap).
- Place the machine in the box noting that the red knobs fit in the slots in the foam packaging.
- Place the small inner box containing power supply and screen into box.
- Release the blue knobs on the footplate and straighten the legs out, put the footplate in the provided bubble wrap and place so middle blue adjust knob is facing upwards. **Observe the pictures attached to show how it should look.**
- Place foam lid into box.
- On top of foam lid leave the power cord and machine documentation.
- Parcel up the box with the tape provided.
- Phone 0161 327 4019 to arrange a collection.



